

## *jota kit 1899* SILICON POLISHERS FOR COMPOSITES





**The economic way to a shiny surface!**




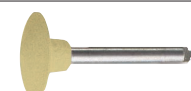
- › Soft silicone polishers
- › Two step system
- › Excellent price-performance ratio
- › Four popular shapes



## jota kit 1899

### SILICON POLISHERS FOR COMPOSITES

<p><b>9121M.RA.050</b> L - 10 mm</p> 	<p><b>Blue: First Step</b></p>
<p><b>9122M.RA.030</b> L - 7.5 mm</p> 	
<p><b>9123M.RA.070</b> L - 10 mm</p> 	
<p><b>9124M.RA.110</b> L - 2.5 mm</p> 	

<p><b>9121F.RA.050</b> L - 10 mm</p> 	<p><b>Yellow: Second Step</b></p>
<p><b>9122F.RA.030</b> L - 7.5 mm</p> 	
<p><b>9123F.RA.070</b> L - 10 mm</p> 	
<p><b>9124F.RA.110</b> L - 2.5 mm</p> 	

⌚ 5000 - 8000 rpm

#### First Step

Blue: smoothing and prepolishing

#### Second Step

Yellow: polishing



Prepolishing of labial surfaces



Prepolishing of occlusal surfaces and humps



Polishing interdental areas with the wheel



Polishing the fissures with the small flame-shaped instruments

JOTA-Dealer

108002.5950.0250 - 07/2012